



Non-contact

# FOREHEAD THERMOMETER

Measuring forehead temperatures

Quick, easy, and reliable!



- 1 Power on device
- 2 Point at centre of forehead 4 cm away
- 3 Press and hold START for one second - a single beep tells you it's done

The non-contact forehead thermometer is pre-set to forehead mode and to display temperature in Celsius. Three short beeps followed by one long beep while taking a forehead reading means that the temperature exceeded 37.5 °C (99.5 °F).

Please wait two seconds between measurements.

**ELECTRONIC TEMPERATURE INSTRUMENTS LTD**

Worthing · West Sussex · BN14 8HQ

01903 202151 · sales@etiltd.com · etiltd.com

## Functions

**Power off device** – Device will auto-off after 60 seconds to save battery life. Or, press and hold Power button for about five seconds. 'Off' will flash before powering off.

**View readings** – With device turned on, press Power button momentarily to see the last reading, which is reading number one. Each subsequent press of the Power button will show up to 25 readings in order. The reading number will flash before displaying the value.

**Mute** – With device turned on, press and hold Power button for three seconds until speaker icon flashes, then release. Repeat steps to unmute.

**Change to surface mode** – With device turned on, press and hold Power button, then press Start to toggle between modes. Device will automatically return to forehead mode after powered off and back on again.

**Change from °C to °F** – Default unit is °C. With device turned off, press and hold Start and the Power button simultaneously for three seconds. Units will change.

## Cautions

- Forehead should be clean, dry, and unobstructed before taking readings.
- Non-contact thermometer is not waterproof. Avoid wet areas and splashes.
- Do not expose device to temperatures above 38 °C (100 °F) or below 10 °C (50 °F).
- Replace batteries when battery icon flashes.
- Store device at room temperature where readings will be taken for best results.
- Avoid drops, bumps, and rough handling that could damage device.
- Sensor lens area is the most delicate part; keep clean and free from debris.
- Consult user manual before cleaning lens.

For guarantee, service, and technical assistance, please contact our technical sales office on 01903 202151 or email [technical@etiltd.co.uk](mailto:technical@etiltd.co.uk)

**ELECTRONIC TEMPERATURE INSTRUMENTS LTD**

Worthing · West Sussex · BN14 8HQ

01903 202151 · [sales@etiltd.com](mailto:sales@etiltd.com) · [etiltd.com](http://etiltd.com)